



Microsoft Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Gyrus ACMI, Inc., ES-IP, ES4K, LiveStream™, nCare™, and VaultStream™ platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite™ HD, ES4K, and ES-IP iOR systems, nCare MX/RX recorders, EasyLink™ Router, LiveStream StatusBoard Engine, and VaultStream systems and its KE Modules.

Also included is information on the latest PHP updates that are approved for the v10.x and up VaultStream systems.

Products: ES-IP and ES4K iOR systems and nCare recorders, connected to a network, VaultStream server software, and the LiveStream StatusBoard Engine.

Date: October 2024

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Gyrus ACMI, Inc. **does not approve** MS updates **KB975560, KB975562, KB2631813, or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues. MedPresence Towers are no longer supported.

Approved Updates:

October 2024 Updates		
Platform	KB Number	Title
VaultStream™ v10.x Updates – Windows Server 2016		
VaultStream 2016	KB5044021	2024-10 Cumulative Update for .NET Framework 4.8 for Windows Server 2016 for x64
VaultStream 2016	KB5044293	2024-10 Cumulative Update for Windows Server 2016 for x64-based Systems
VaultStream 2016	KB890830	Windows Malicious Software Removal Tool x64 - v5.129
VaultStream™ v10.4 Updates – Windows Server 2019		
VaultStream 2019	KB5044089	2024-10 Cumulative Update for .NET Framework 3.5, 4.7.2, and 4.8 for Windows Server 2019 for x64
VaultStream 2019	KB5044277	2024-10 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems
VaultStream 2019	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.419.402.0) - Current Channel Broad
VaultStream 2019	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.419.401.0) - Current Channel Broad
VaultStream 2019	KB2538243	Security Update for Microsoft Visual c++ 2008 Service Pack 1 Redistributable Package
VaultStream 2019	KB890830	Windows Malicious Software Removal Tool x64 - v5.129
VaultStream v11.x Updates – Windows Server 2022		
VaultStream 2022	KB5044281	2024-10 Cumulative Update for Microsoft server operating system version 21H2 for x64-based Systems
VaultStream 2022	KB5044099	2024-10 Cumulative Update for .NET Framework 3.5,4.8 and 4.8.1 for Microsoft server operating system version 21H2 for x64
VaultStream 2022	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (version 1,419.402.0) - Current Channel Broad
VaultStream 2022	KB890830	Windows Malicious Software removal Tool x64 -v5.129

nCare™ MX v10.2.3		
nCare MX	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.1963.0)
nCare MX	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.3887.0)
nCare MX	KB5044293	2024-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
nCare MX	KB5044021	2024-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
nCare™ MX v10.4		
nCare MX	KB5044293	2024-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
nCare MX	KB5044021	2024-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
nCare MX	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.1963.0)
nCare MX	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.3887.0)
ES4K v10.4 ES4K Controller (a.k.a. BlizCon)		
ES4K Controller	KB5044293	2024-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
ES4K Controller	KB5044021	2024-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
ES4K Controller	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.3887.0)
ES4K Controller	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.1963.0)
ES4K v10.4 ES4K Video Manager (a.k.a. nCare RX2)		
ES4K Video Manager	KB5044293	2024-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
ES4K Video Manager	KB5044021	2024-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
ES4K Video Manager	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.3887.0).
ES4K Video Manager	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.1963.0).
ES4K v10.3 LiveStream™ StatusBoard 2		
LiveStream StatusBoard 2	KB5044293	2024-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems.
LiveStream StatusBoard 2	KB5044021	2024-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.3887.0).
LiveStream StatusBoard 2	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.1963.0).

ES-IP iOR system		
ES-IP	KB5044090	2024-10 cumulative Update for .NET Framework 3.5, 4.8 and 4.8.1 for Windows 10 Version 21H2 for x64
ES-IP	KB5044273	2024-10 Cumulative Update for Windows 10 Version 21H2 for x64-based Systems
ES-IP	KB5045998	2024-10 .NET 6.0.35 Security Update for x64 Client
ES-IP	KB890830	Security Intelligence Update for Microsoft Defender Antivirus - (Version
ES-IP	KB2267602	Windows Malicious Software Removal Tool x64 - v5.129
MAJ-2533 (UCES-4 medical device controller)		
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.419.633.0)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.419.603.0)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.419.587.0)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.419.569.0)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.419.547.0)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.419.507.0)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.419.478.0)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.419.415.0)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.419.396.0)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.419.379.0)
MAJ-2533	KB2267602	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.419.369.0)
MAJ-2533	KB5044277	2024-10 cumulative Update for Windows 10 Version 1809 for x64-based System
MAJ-2533	KB5044089	2024-10 cumulative Update for .NET Framework 3.5, 4.8 and 4.8 for Windows
MAJ-2533	KB890830	Windows Malicious Software Removal Tool x64 - v5.129
CV Connectivity with KE v3.1 (VaultStream with Interfaces License)		
KE v3.1	KB5043064	2024-09 Cumulative Update for Windows 10 Version 22H2 for x64-based Systems
KE v3.1	KB5043051	2024-09 Cumulative Update for Windows Server 2016 for x64-based Systems
KE v3.1	KB5043124	2024-09 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KE v3.1	KB5043050	2024-09 Cumulative Update for Windows Server 2019 (1809) for x64- based Systems
KE v3.1	KB5042881	2024-09 Cumulative Update for Microsoft server operating system version 21H2 for x64-based Systems
KE v3.1	KB890830	Windows Malicious Software Removal Tool x64 - v5.128



V10.2.2 was Tested with PHP version 7.3.33
V10.4 was Tested with PHP version 7.4.30
PHP 7.3.33 is available upon request for v10.2.2 VaultStreams only.
PHP 7.4.33 is available upon request for v10.4 VaultStreams only.
PHP V8.1.19 is available upon request v10.7 VaultStreams only.
PHP V8.3.8 is available upon request v10.7 VaultStreams only.
PHP V8.3.9 is available upon request v10.7 VaultStreams only

For further questions concerning this notice, please call Technical Support at 1-800-848-9024

Microsoft and Windows are trademarks of the Microsoft group of companies.



Gyrus ACMI, Inc.

One 9600 Louisiana Ave. North, Brooklyn Park, MN 55445 U.S.A.
www.olympusamerica.com/olympus-systems-integration